
XL t(4;11) AFF1/KMT2A DF

Translocation/ Dual Fusion Probe

XL t(4;11) AFF1/KMT2A DF is designed as a dual fusion probe. The orange labeled probe hybridizes to the AFF1 gene region at 4q21, the green labeled probe hybridizes to the KMT2A gene region at 11q23.

Clinical Details

Chromosomal rearrangements of the KMT2A (lysine methyltransferase 2A) gene, formerly MLL (mixed lineage leukemia), are associated with various hematological disorders. Most patients suffer from acute myeloid leukemia (AML) or acute lymphoblastic leukemia (ALL), while only a minority develops mixed lineage leukemia (MLL). Several chromosomal aberrations involving the KMT2A gene have been identified. However, the majority of leukemias result from translocations leading to KMT2A fusions. More than 90 KMT2A translocation partner genes fused to the 5'-KMT2A portion have been identified. The most common translocation partners in KMT2A associated leukemia are AFF1, MLLT3, MLLT1, MLLT10, ELL and AFDN, described here in the order of their frequency. The fusion of KMT2A and AFF1 (AF4/FMR2 family member 1) is the most frequent KMT2A translocation observed in ALL cases. About 60% of ALL patients carrying KMT2A rearrangements are characterized by the KMT2A-AFF1 fusion gene. Infant patients mainly have breakpoints in intron 11 of KMT2A, whereas adult patients have a tendency for breakpoints within intron 9. Oncogenic KMT2A-AFF1 fusions have been implicated in the DOT1L and SEC transcriptional complexes. AFF1 plays an important role in the regulation of RNA polymerase II-dependent transcription, elongation and chromatin remodeling mediated by histone methylation. AFF1 is the docking platform for several components of the DOT1L complex consisting of DOT1L, a H3K79 methyltransferase, MLLT3, MLLT1, and MLLT10, whose genes are also common translocation partners of KMT2A. Mouse studies revealed increased and extended histone methylation signatures in the presence of KMT2A-AFF1 and reciprocal AFF1-KMT2A fusions. DOT1L inhibitors are promising candidates for clinical treatment which are currently being evaluated.

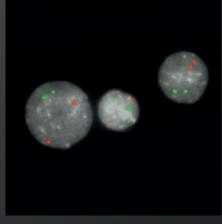

Order No.:

D-5131-100-OG

XL t(4;11) AFF1/KMT2A DF hybridized to lymphocytes. Three normal interphases are shown.


Clinical Applications

Acute Lymphoblastic Leukemia (ALL)

Literature

- De Braekeleer et al (2011) Mol Oncol 5: 555-563
- Meyer et al (2013) Leukemia 27:2165-2176
- Burns et al (2018) Hematology (7th ed) Ch 64: Pathobiology of acute lymphoblastic leukemia:1005–1019.e11



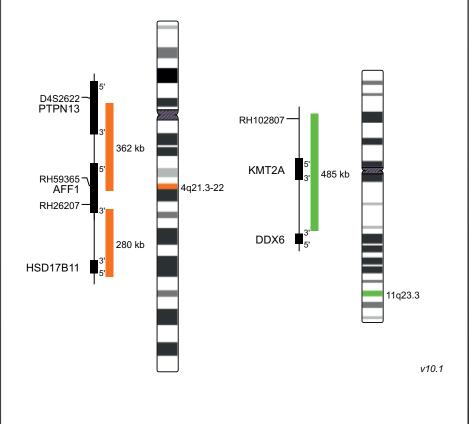

 Normal Cell: Two green (2G) and two orange (2O) signals.

 Aberrant Cell (typical result): One green (1G), one orange (1O), and two greenorange colocalization/fusion signals (2GO) resulting from a reciprocal translocation between the respective loci.

MetaSystems Probes

EUROPE & RUSSIA

Germany, Altlussheim info@metasystems-probes.com

Italy, Milano info@metasystems-italy.com

Russia, Moscow info@metasystems.su

AMERICA

USA, Newton info@metasystems.org

Argentina, Buenos Aires info@metasystems-latam.com

ASIA & INDIA

China, Hong Kong info@metasystems-asia.com

China, Taizhou info@metasystems-china.com

India, Bangalore info@metasystems-india.com

FACTSHEET



